


Fast Fit® Sizing Jig

Printable Version

CASCADE®
The daFO People®

Left Foot | Top

Cut/fold along dotted line. Connect Bottom portion of Left Foot here.

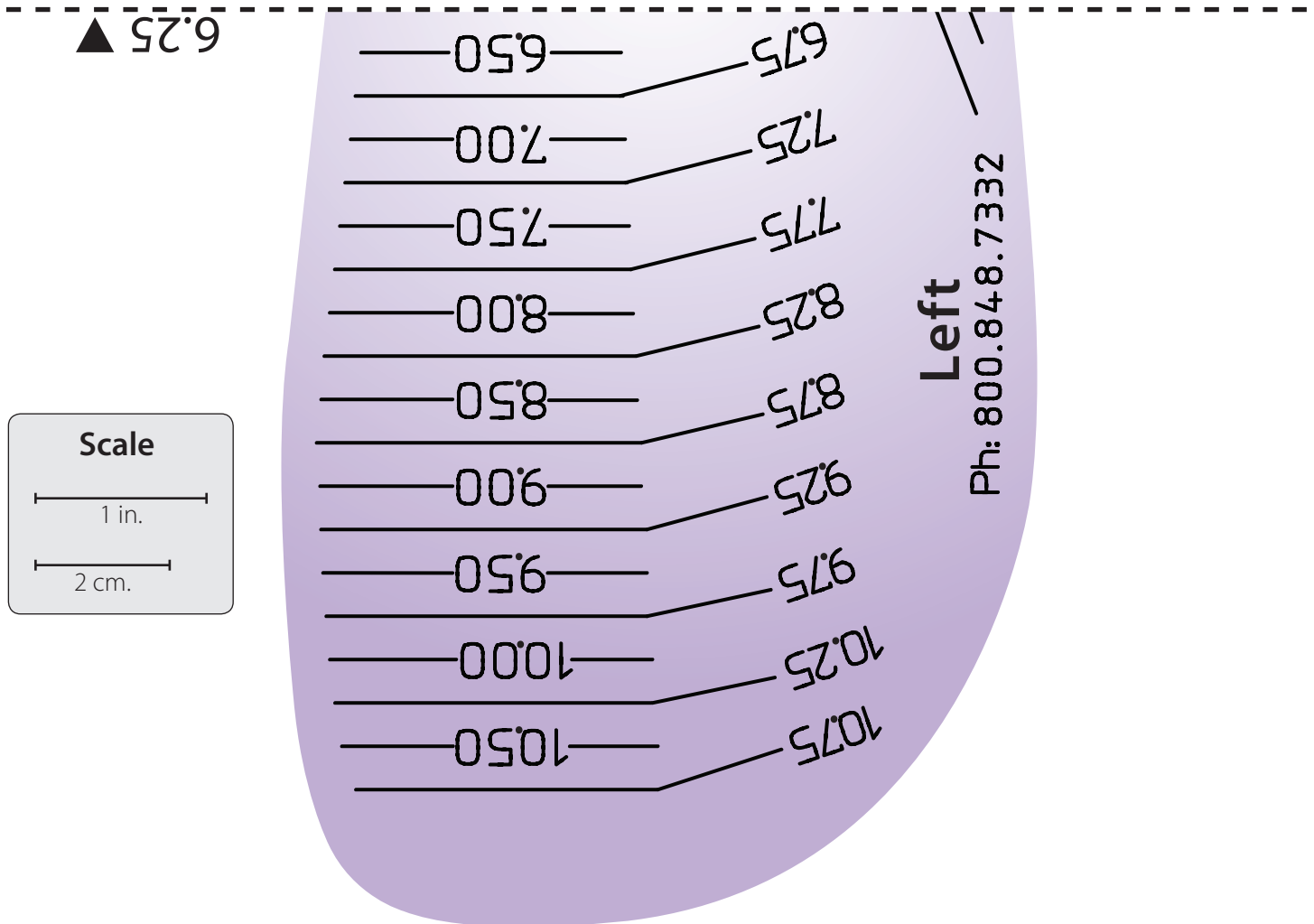

Scale

1 in.

2 cm.

1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Under Page Scaling, select None. (DO NOT select Fit to Printable Area or Fit to Page. If there's a percentage option, make sure it's set at 100%.)
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.
**Look from the side and make sure the foot doesn't hang over either of these edges.*
2. Choose the length that will allow ¼ to ½ in. (.64 to 1.3 cm.) for growth.
3. If the outside edge of the foot crosses the Narrow line, choose the Wide option.


Printable Sizing Jig — Left Top rev.01 (Nov 11)

Fast Fit® Sizing Jig

Printable Version

Left Foot | Bottom

Cut/fold along dotted line. Connect Top portion of Left Foot here.

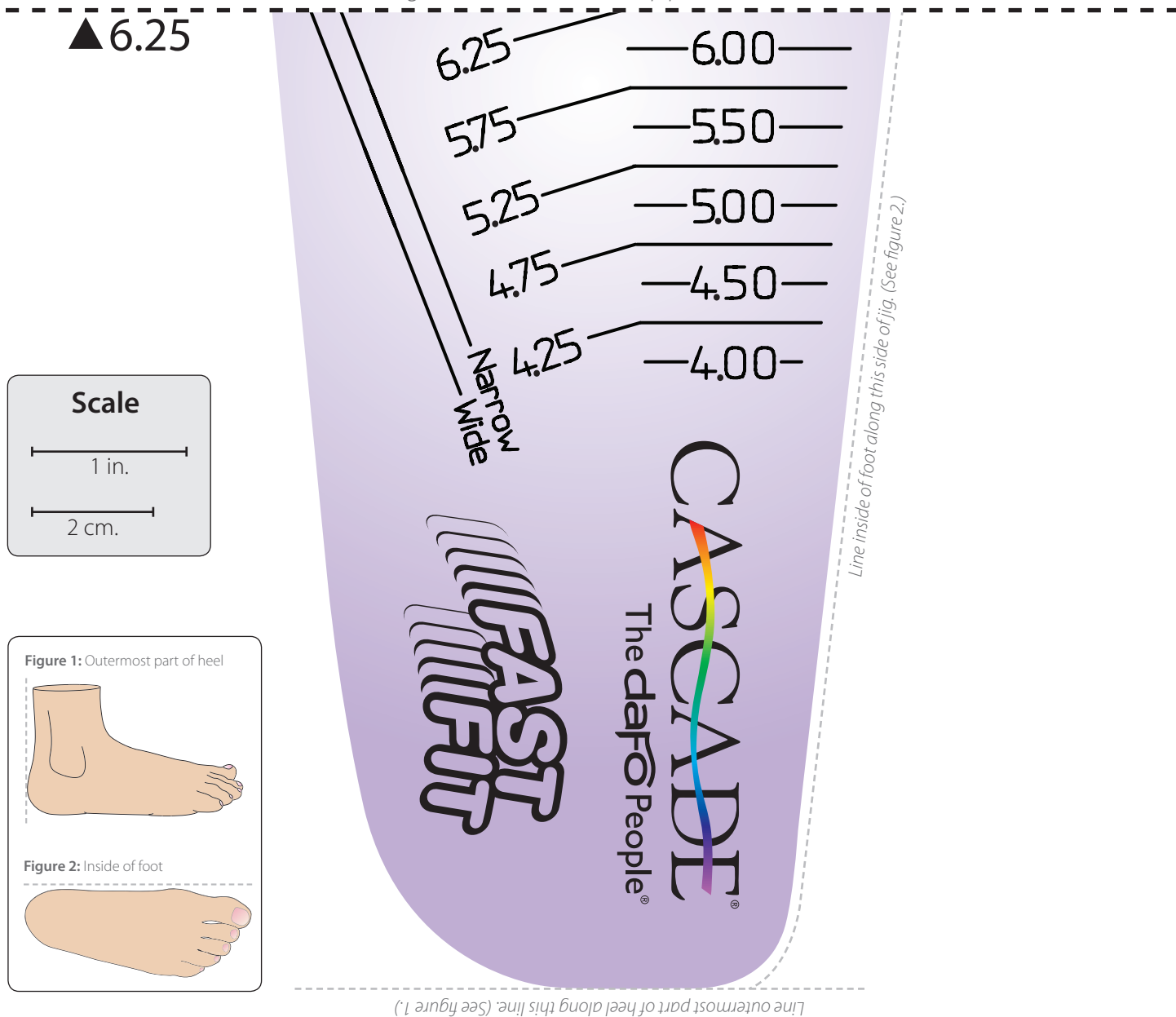


Fast Fit® Sizing Jig

Printable Version

CASCADE®
The daFO People®

Right Foot | Top

Cut/fold along dotted line. Connect Bottom portion of Right Foot here.

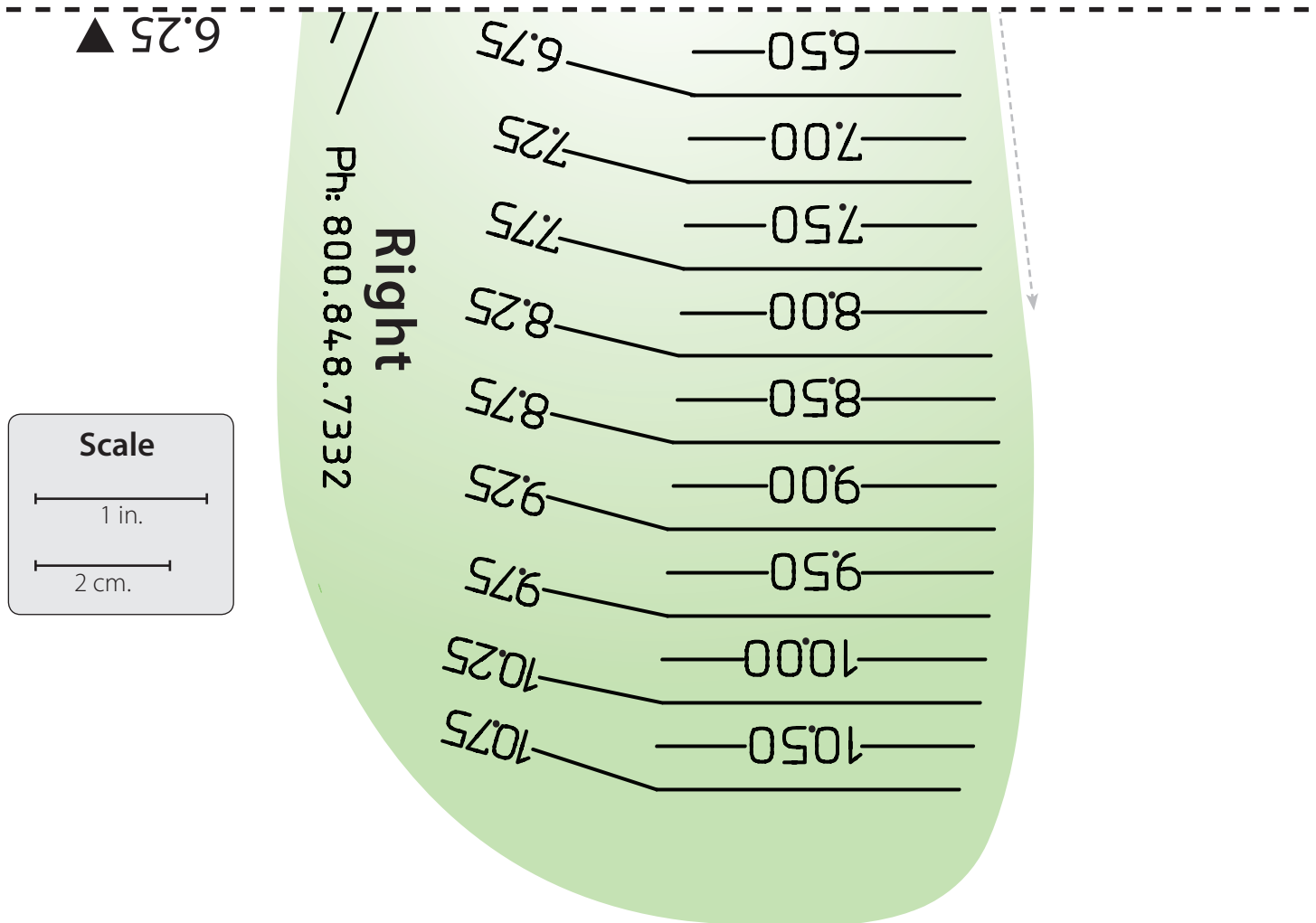

Scale

1 in.

2 cm.

1 Print

To ensure accurate scaling and proper sizing, use the following printer settings:

1. With the Printable Sizing Jig pdf open in Adobe Reader, go to the File menu.
2. Click the Print sub-menu item.
3. Under Page Scaling, select None. (DO NOT select Fit to Printable Area or Fit to Page. If there's a percentage option, make sure it's set at 100%.)
4. Print.
5. After printing, verify that the scale on this sheet is accurate with a standard ruler.

2 Assemble

1. Follow the printing instructions (to the left). If the scaling is accurate, continue on.
2. Cut/fold along the dotted lines on the jigs' Top and Bottom portions.
3. Tape the Top and Bottom portions of the Left and Right sizing jigs at their corresponding dotted lines.

3 Size

1. Make sure the patient is wearing socks and have him/her stand on the jig. Align the outermost part of the heel and the inside of the foot with the designated lines on the jig.
**Look from the side and make sure the foot doesn't hang over either of these edges.*
2. Choose the length that will allow ¼ to ½ in. (.64 to 1.3 cm.) for growth.
3. If the outside edge of the foot crosses the Narrow line, choose the Wide option.

Printable Sizing Jig — Right Top rev.01 (Nov 11)

Fast Fit® Sizing Jig

Printable Version

Right Foot | Bottom

Cut/fold along dotted line. Connect Top portion of Right Foot here.

