

# Using the Super Heel Seating Strap for the JumpStart® Kangaroo®

The Super Heel Seating strap is an instep strap that helps ensure proper placement of the heel within the JumpStart Kangaroo. This strap also holds an instep pad to cushion the foot from the strap.

1



Loosen the instep strap by pulling the instep pad toward you.

2



With hip, knee and ankle flexed, slide your patient's foot into the brace under the instep pad. Make sure the heel is well seated.

3



Tuck the instep pad inside the brace and tighten the instep strap by pulling the end of the strap away from the brace.

4



Keeping tension, secure the instep strap to its corresponding tab. The strap should be tight enough to hold the heel snugly into the heel cup.



For more information or questions, ask Customer Service:  
**800.848.7332** | [customerservice@dafo.com](mailto:customerservice@dafo.com)