

Applying the JumpStart Toe Rise Pad (with optional toe abduction strap)



Here's how to precisely position the toe rise pad on your JumpStart® for best comfort and support. The toe pad may include a toe abduction strap.


1



With the patient's socked foot in the JumpStart and straps secured, lift the toes slightly and slide the toe rise pad into place without removing the protective paper.

2

Position the toe rise pad so that a small gap exists between the metatarsal heads and the pad. *A gap too large or none at all causes discomfort. When positioning, consider the weight-bearing size and expected growth of the foot.*

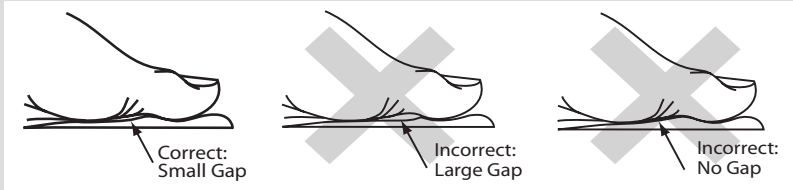

3



Marking for Toe Rise Pad Placement

- **If the toe rise pad is longer than the JumpStart:**
Trace the front edge of the brace along the underside of the pad to make a trimming reference line.
- **If the JumpStart is longer than the toe rise pad:**
Draw a line on the plastic using the distal end of the pad as your guide.

4



Marking for Final Trim Line

Mark the top of the toe rise pad along the toe shelf for final trimming. *A thumb's width is often a good measure of anticipated growth for children.*

5



Trimming for Toe Rise Pad Placement

Remove the toe rise pad and foot from the brace and trim to the marked line (on either the plastic or the pad).

6



Remove the protective paper from the toe rise pad and attach the pad to the DAFO.

7



Trimming to Final Trim Line

Trim to the final trimming line along the toe shelf.

↳ If your toe pad includes a toe abduction strap, please continue to steps 8 – 10.

8



With the patient's foot back in the brace and straps secured, test the toe strap placement by finding your patient's best corrected toe position.

9



Attach the hook tab onto the bottom of the brace so that it aligns with the toe strap placement.

10



Pull the toe strap over the toe and attach it to the hook tab. Trim the strap at an angle to desired length.



For more information or questions, ask Customer Service:
800-848-7332 • customerservice@dafo.com

Cascade Dafo, Inc.

1360 Sunset Ave
Ferndale, WA 98248

ph: 800.848.7332
fax: 877.856.2160

intl: + 00 1 360 543 9306
www.cascadedrafo.com

Turn Over →